



**INSTALLATION INSTRUCTIONS
S-SERIES INBOARD ROLLSTOP RETROFIT
KITS: 01025/01026/01029/01030**

- Read and understand supplemental instructions completely before installation is attempted. Refer to the Service/Owner Manual for additional installation instructions and safety precautions.
- Check contents of kit. Immediately notify Ricon of any missing parts. Claims for damages that have occurred during shipment must be made to the shipper.

TABLE OF CONTENTS	<u>PAGE</u>
I. INTRODUCTION	1
II. TOOLS REQUIRED	2
III. DISASSEMBLY PROCEDURE	2
IV. ASSEMBLY PROCEDURE	3

I. INTRODUCTION

- The purpose of this kit is to retrofit an existing S-Series lift rollstop with a spring loaded rollstop.
- The original inboard rollstop must be returned to Ricon Corp. Use packing materials from the replacement parts. Be sure R.M.A. number is placed on the outside of the box.

II. TOOLS REQUIRED

7/16" Wrenches	1/2" wrench	3/16" and 5/32" Hex Wrenches
----------------	-------------	------------------------------

III. DISASSEMBLY PROCEDURE

- REMOVE INBOARD ROLLSTOP FROM PLATFORM:

1. Remove Delrin Guides located on the bottom side corners of the rollstop **Figure [1]**.
2. If your lift is equipped with a Sto-Loc, remove the base Latch Catch located on the bottom of the rollstop **Figure [2]**.
3. Disconnect both cables from both sides of the rollstop **Figure [3]**.
4. Remove the Inboard Rollstop weld assembly from the lift **Figure [4]**.



FIGURE [1]: DELRIN GUIDE REMOVAL



FIGURE [2]: STO-LOC ON THE BOTTOM OF ROLLSTOP

IV. ASSEMBLY PROCEDURE

- **PLACING NEW INBOARD ROLLSTOP ASSEMBLY:**

1. Attach original Delrin Guides to the bottom side corners of the rollstop **Figure [1]**.
2. Attach new rollstop to the platform using new bolts and nuts in supplied kit **Figure [4]**.

NOTE: Be sure the hinge is connected to the top side of the platform.

3. Reconnect cables to sides of the rollstop **Figure [3]**.
4. If your lift is equipped with a Sto-loc, reconnect the Base Latch Catch to the rollstop. Be sure the edge with a cut in corner is facing away and down from the platform **Figure [2]**.
 - a. Fold the lift.
 - b. Be sure the Base Catch Latch drops just behind the base catch. If it does not, adjust the latch in the appropriate direction.
 - c. Unfold and Fold lift. If the Base Latch Catch is not properly adjusted, repeat procedures starting from step 4.b.



FIGURE [3]: CABLES ON BOTH SIDES OF ROLLSTOP



FIGURE [4]: ROLLSTOP

NOTES