



**INNOVATION IN MOBILITY™**

***ECLIPSE®  
F8000 (A) Series  
Export Use  
Wheelchair and Standee Lift***

[PRINT ENTIRE MANUAL](#)

[TABLE OF CONTENTS](#)

***Service/Owner Manual***

(To be retained by Owner After Installation by Authorized RICON Dealer)

**This RICON product must be installed and serviced only by authorized RICON dealers.**

**The authorized RICON dealer must refer to this manual for installation instructions.**

**The owner must refer to this manual for operating instructions, then retain it for future reference by authorized RICON dealers who perform service and repairs.**

Customer Name: _____
Installing Dealer: _____
Date Installed: _____
Serial Number: _____

## TRADEMARK ACKNOWLEDGMENTS

©Copyright 1997 thru 1998 by Ricon® Corporation. All rights reserved. No part of this publication may be reproduced, transmitted, stored in a retrieval system, or translated into any language, in any form or by any means, mechanical, optical, electronic, recording, or otherwise, without prior consent from Ricon. Ricon reserves the right to revise this manual and to make changes to any or all parts at any time, without obligation to notify any person or entity of such revisions and changes.

The following are trademarks of Ricon®:

<i>Accessibility with Style</i> ®	Mirage™	Trimway™
<i>Activan</i> ®	Ricon®	UNI-lite™
Clearway™	S-Series™	
Eclipse®	Spirit Swing Lift®	

No investigation has been made of common-law trademark rights in any word. Words that are known to have current registrations are shown as such. The inclusion or exclusion of any word is not, however, an expression of the developer's opinion as to whether or not it is subject to proprietary rights, nor is it to be regarded as affecting the validity of any trademark. The following are noted trademarks:

TRADEMARK	TRADEMARK
Curtisol®	omni-TITE®
loc-TITE®	Teflon®
molex®	WD-40®

## RECORD OF CHANGE OR REVISION

This listing contains a record of all page changes and/or revisions added to or to be added into this manual, since the last revision. Not all of the listed pages contain changes but are included for ease of installation. The arrows (» or °) found on the outer edge of the page(s) indicate the revised or changed area, section, or paragraph of the page.

REV.	PAGE(S)	DESCRIPTION OF CHANGE	ECR/ECO
32DF8AE0.C 06/19/98	ALL	Various revisions and updates.	001575/3240
<b>END OF LIST</b>			

<b>TABLE OF CONTENTS</b>	<b>PAGE</b>
<b>I. INTRODUCTION</b> .....	<b>1-1</b>
RICON 1-YEAR LIMITED WARRANTY .....	1-2
DEALERS INFORMATION .....	1-3
GENERAL SAFETY PRECAUTIONS .....	1-3
PRODUCT TERMINOLOGY .....	1-4
<b>II. INSTALLATION</b> .....	<b>2-1</b>
<b>A. MECHANICAL</b> .....	<b>2-1</b>
1. LIFT LOCATION .....	2-1
2. LIFT MOUNTING .....	2-1
3. HYDRAULIC POWER UNIT .....	2-3
a. Unit Mounting .....	2-3
b. Unit-to-Cylinder Connection .....	2-3
<b>B. ELECTRICAL</b> .....	<b>2-4</b>
1. INTERLOCK WIRE .....	2-4
2. WIRING FOR OPTIONAL PUMP ENCLOSURE .....	2-5
3. MAIN WIRING .....	2-5
<b>C. FINAL ADJUSTMENTS</b> .....	<b>2-7</b>
1. FLOOR LEVEL TRAVEL LIMIT SETTING .....	2-7
2. BRIDGEPLATE ADJUSTMENT .....	2-9
3. LIFT STOW LEVEL SETTING .....	2-10
4. DEPLOY LOCK SETTING .....	2-11
5. STOW LOCK SETTING .....	2-12
<b>D. VERIFY INSTALLATION</b> .....	<b>2-13</b>
<b>E. CUSTOMER ORIENTATION</b> .....	<b>2-14</b>
<b>III. OPERATING INSTRUCTIONS</b> .....	<b>3-1</b>
<b>A. SAFETY PRECAUTIONS</b> .....	<b>3-1</b>
<b>B. DAILY SAFETY CHECK</b> .....	<b>3-3</b>
<b>C. LIFT FUNCTIONS</b> .....	<b>3-3</b>
<b>D. CONTROLS AND INDICATORS</b> .....	<b>3-4</b>
1. CONTROL PENDANT .....	3-4

2.	ELECTRICAL CONTROLS AND INDICATORS .....	3-5
a.	Main Circuit Breaker .....	3-5
b.	Control System Circuit Breaker .....	3-5
3.	MANUAL BACKUP PUMP .....	3-6
a.	To Raise Platform Manually .....	3-6
b.	To Lower Platform Manually .....	3-7
<b>E.</b>	<b>LIFT OPERATION .....</b>	<b>3-8</b>
1.	TO ENTER VEHICLE .....	3-9
2.	TO EXIT VEHICLE .....	3-9
3.	TO STOW LIFT .....	3-10
4.	MANUAL OPERATION .....	3-10
a.	To Manually Deploy the Lift .....	3-10
b.	To Manually Lower or Raise the Platform .....	3-11
c.	To Manually Stow the Lift .....	3-11
5.	EMERGENCY STOW PROCEDURE .....	3-12
<b>IV.</b>	<b>MAINTENANCE .....</b>	<b>4-1</b>
<b>A.</b>	<b>MAINTENANCE SCHEDULE .....</b>	<b>4-1</b>
<b>B.</b>	<b>TROUBLESHOOTING GUIDES .....</b>	<b>4-3</b>
1.	LIFT TROUBLESHOOTING GUIDE .....	4-3
2.	LIFT OPERATIONAL TROUBLESHOOTING .....	4-5
<b>C.</b>	<b>POWER CUT-OFF INTERLOCK INDICATOR DIAGNOSTICS .....</b>	<b>4-6</b>
<b>D.</b>	<b>ELECTRICAL WIRING DIAGRAMS .....</b>	<b>4-6</b>
1.	Color Codes .....	4-6
2.	Connectors .....	4-7
3.	Labels .....	4-7
4.	Symbols .....	4-7
5.	Wiring Diagrams .....	4-7
<b>E.</b>	<b>HYDRAULIC POWER UNIT FLUID FLUSH AND RENEWAL .....</b>	<b>4-10</b>
<b>F.</b>	<b>BLEEDING THE HYDRAULIC SYSTEM .....</b>	<b>4-11</b>
<b>V.</b>	<b>PARTS DRAWINGS AND LISTS .....</b>	<b>5-1</b>
<b>APPENDIX 1.</b>	<b>LIFT SPECIFICATIONS .....</b>	<b>5-23</b>

**LIST OF FIGURES****PAGE**

FIGURE [1-1]: LIFT REFERENCES .....	1-4
FIGURE [2-1]: LIFT MOUNTING ATTACHMENT POINTS .....	2-1
FIGURE [2-2]: LIFT MOUNTING FASTENER .....	2-2
FIGURE [2-3]: INTERLOCK WIRING .....	2-4
FIGURE [2-4]: WIRING FOR OPTIONAL PUMP ENCLOSURE .....	2-5
FIGURE [2-5]: TRAVEL LIMIT ADJUSTMENT .....	2-7
FIGURE [2-6]: TRUNION REMOVAL .....	2-8
FIGURE [2-7]: BRIDGEPLATE ADJUSTMENT .....	2-9
FIGURE [2-8]: STOW LEVEL ADJUSTMENT .....	2-10
FIGURE [2-9]: DEPLOY LOCK ADJUSTMENT .....	2-11
FIGURE [2-10]: STOW LOCK HOOK ADJUSTMENT .....	2-12
FIGURE [2-11]: DECAL LOCATIONS AND PART NUMBERS .....	2-14
FIGURE [3-1]: UNSAFE VEHICLE SLOPING .....	3-1
FIGURE [3-2]: PLATFORM POSITIONS .....	3-3
FIGURE [3-3]: CONTROL PENDANT .....	3-4
FIGURE [3-4]: MAIN CIRCUIT BREAKER .....	3-5
FIGURE [3-5]: CONTROL SYSTEM CIRCUIT BREAKER .....	3-5
FIGURE [3-6]: STANDARD PUMP ENCLOSURE .....	3-6
FIGURE [3-7]: PLATFORM MANUAL RAISING .....	3-6
FIGURE [3-8]: PLATFORM MANUAL LOWERING .....	3-7
FIGURE [3-9]: STOW LOCK HANDLE .....	3-8
FIGURE [3-10]: PLATFORM LOCK BAR .....	3-8
FIGURE [3-11]: DEPLOY LOCK HANDLE .....	3-10
FIGURE [4-1]: MOLEX® CONNECTORS .....	4-7
FIGURE [4-2]: ELECTRICAL DIAGRAM SYMBOLS .....	4-7
FIGURE [4-3]: WIRING DIAGRAM W/STANDARD PUMP ENCLOSURE .....	4-8
FIGURE [4-4]: WIRING DIAGRAM W/OPTIONAL PUMP ENCLOSURE .....	4-9
FIGURE [5-1]: MONARCH HYDRAULIC POWER UNIT .....	5-2
FIGURE [5-2]: PUMP INTERIOR (STANDARD ENCLOSURE) MOUNTING ASSEMBLY .....	5-4

FIGURE [5-3]: OPTIONAL PUMP ENCLOSURE ASSEMBLY ..... 5-6

FIGURE [5-4]: LIFT ENCLOSURE ASSEMBLY ..... 5-10

FIGURE [5-5]: PLATFORM ASSEMBLY ..... 5-14

FIGURE [5-6]: TRAVELING FRAME ASSEMBLY ..... 5-18

FIGURE [5-7]: CARRIAGE ASSEMBLY ..... 5-20

**LIST OF TABLES** **PAGE**

---

TABLE [1-1]: F8A SERIES LIFT TERMINOLOGY ..... 1-5

TABLE [2-1]: LIFT MOUNTING BRACKET LOAD CAPACITIES ..... 2-2

TABLE [2-2]: HYDRAULIC POWER UNIT BRACKET LOAD CAPACITIES ..... 2-3

TABLE [3-1]: LIFT FUNCTIONS ..... 3-3

TABLE [4-1]: MAINTENANCE SCHEDULE ..... 4-1

TABLE [4-2]: TROUBLESHOOTING GUIDE ..... 4-4

TABLE [4-3]: OPERATIONAL TROUBLESHOOTING GUIDE ..... 4-5

TABLE [4-4]: INTERLOCK INDICATOR TROUBLESHOOTING GUIDE ..... 4-6

TABLE [4-5]: COLOR CODE DEFINITIONS ..... 4-6