



INNOVATION IN MOBILITY™

PSL-10
STEP ELEVATOR
and
WHEELCHAIR LIFT

[*PRINT ENTIRE MANUAL*](#)

[*TABLE OF CONTENTS*](#)

OPERATOR MANUAL



07/02/01

32DSL101.B

© 95-2002 RICON CORPORATION

All Rights Reserved

U.S. Patent no. 4,606,433
Other U.S and Foreign Patents Pending
Printed in the United States of America

**ONLY AUTHORIZED RICON SERVICE TECHNICIANS
MAY SERVICE THIS RICON PRODUCT.**

**PRODUCT USERS MUST REFER TO THIS MANUAL
FOR OPERATING AND GENERAL MAINTENANCE
INSTRUCTIONS.**

**RETAIN THIS MANUAL IN THE VEHICLE FOR FUTURE
REFERENCE.**

Customer Name: _____

Installing Dealer: _____

Date Installed: _____

Serial Number: _____

REVISION RECORD

REVISION	PAGES	DESCRIPTION OF CHANGE	ECR/ECO
32DSL101. B 07/02/01	All	Changed to curved arms on iso throughout manual. Changed "dealer" to "service technician".	3740/4773
	1-3	Changed "dealer" to "shipment".	
	2-1 & 2-5	Added "UNLOCK" function to pendant.	
	2-2 & 2-3	Updated drawings, deleted item 3, added "stow lock manual release cable".	
	2-5, 3-3, 3-7 & 3-8	Rewrote "stow lock release" references to current configuration.	
	3-7 & 3-10	Added Figure "Close Release Valve"	
	4-1	Updated Daily Safety Check Table	
	4-2	Deleted decal 17736, changed decal 17631 to 19552	

TABLE OF CONTENTS

<i>Chapter:</i>	<i>Page</i>
I. INTRODUCTION	1-1
A. RICON PRODUCT SUPPORT	1-1
B. STEP-LIFT WARRANTY	1-2
C. SHIPMENT INFORMATION	1-3
D. GENERAL SAFETY PRECAUTIONS	1-3
II. DESCRIPTION	2-1
A. STEP-LIFT FUNCTIONS AND POSITIONS	2-1
B. MAJOR STEP-LIFT COMPONENTS	2-2
C. CONTROLS	2-5
1. CONTROL PENDANT	2-5
2. EMERGENCY OVERRIDE SWITCH	2-5
3. MANUAL STOW-LOCK RELEASE CABLE	2-5
4. PLATFORM SECTION RELEASE HANDLE	2-6
5. FRONT SECTION HANDLE AND ROLLSTOP HANDLE	2-6
6. MANUAL BACK-UP PUMP	2-6
III. OPERATING INSTRUCTIONS	3-1
A. SAFETY PRECAUTIONS	3-1
B. DAILY SAFETY CHECK	3-2
C. STEP-LIFT OPERATION	3-2
1. NORMAL STEP OPERATION	3-3
2. NORMAL WHEELCHAIR LIFT OPERATION	3-4
3. MANUAL OPERATION	3-7
IV. MAINTENANCE	4-1
A. DAILY SAFETY CHECK	4-1
B. STEP-LIFT DECALS	4-2

LIST OF FIGURES

<i>Figure</i>	<i>Page</i>
2-1 CONTROL PENDANT	2-1
2-2 STEP/PLATFORM POSITIONS	2-1
2-3 STEP ELEVATOR COMPONENTS	2-2
2-4 WHEELCHAIR LIFT COMPONENTS.....	2-3
2-5 CONTROL PENDANT	2-5
2-6 LEFT HAND TOWER WITH EMERGENCY OVERRIDE SWITCH	2-5
2-7 STOW-LOCK RELEASE CABLE LOCATION	2-5
2-8 MIDDLE SECTION RELEASE T-HANDLE	2-6
2-9 FRONT SECTION HANDLE AND ROLLSTOP HANDLE	2-6
2-10 MANUAL BACK-UP PUMP.....	2-6
3-1 DO NOT USE STEP-LIFT WHEN VEHICLE IS ON SLOPED GROUND.....	3-1
3-2 UNFOLD STEP WITH PALM UP.....	3-4
3-3 UNLATCH FRONT PLATFORM SECTION	3-4
3-4 UNFOLD FRONT PLATFORM SECTION	3-5
3-5 RAISE OUTER ROLLSTOP	3-5
3-6 LOWER OUTER ROLLSTOP	3-5
3-7 LOWER OUTER ROLLSTOP	3-6
3-8 RAISE OUTER ROLLSTOP	3-6
3-9 STORE OUTER ROLLSTOP	3-6
3-10 RAISE STEP-LIFT ABOUT 4"	3-7
3-11 CLOSE RELEASE VALVE	3-7
3-12 UNFOLD STEP WITH PALM UP.....	3-7
3-13 UNLATCH FRONT PLATFORM SECTION	3-8
3-14 UNFOLD FRONT PLATFORM SECTION	3-8
3-15 RAISE OUTER ROLLSTOP	3-8
3-16 RELEASE STOW-LOCK.....	3-9
3-17 OPEN RELEASE VALVE	3-9
3-18 LOWER OUTER ROLLSTOP	3-9
3-19 RAISE STEP-LIFT ABOUT 4"	3-10
3-20 CLOSE RELEASE VALVE	3-10
3-21 UNFOLD STEP WITH PALM UP.....	3-10
3-22 UNLATCH FRONT PLATFORM SECTION	3-11
3-23 UNFOLD FRONT PLATFORM SECTION	3-11
3-24 RAISE OUTER ROLLSTOP	3-11
3-25 PULL MANUAL RELEASE CABLE.....	3-12
3-26 OPEN RELEASE VALVE	3-12
3-27 RAISE OUTER ROLLSTOP	3-12
4-1 DECAL LOCATIONS.....	4-2

LIST OF TABLES

<i>Table</i>		<i>Page</i>
2-1	STEP-LIFT FUNCTIONS.....	2-1
2-2	STEP-LIFT COMPONENTS.....	2-4
4-1	DAILY SAFETY CHECK.....	4-1

This page intentionally left blank.

[-BACK TO FRONT COVER-](#)

[-GO TO NEXT CHAPTER-](#)