



Innovation in Mobility

Activan®

Accessibility With Style®

for

Caltrans

General Motors Minivan Conversion

SERVICE MANUAL

Servicing by authorized Ricon service technicians is required during the warranty period.

Ricon service personnel must refer to this manual for operating and general maintenance instructions.

REVISION RECORD

REVISION	DATE	PAGE	DESCRIPTION OF CHANGE	ECO
32DV0004. A	04/27/00	NA	Initial release for production.	NA

TABLE OF CONTENTS

<u>Chapter</u>	<u>Page</u>
I. INTRODUCTION	1-1
LIMITED WARRANTY	1-2
GENERAL SAFETY PRECAUTIONS	1-3
II. ACTIVAN FEATURES	2-1
FOLDING RAMP	2-1
REAR STORAGE BIN AND SPARE TIRE MOUNT	2-1
BACK-UP RAMP (OPTIONAL)	2-2
Q'STRAIT® WHEELCHAIR RESTRAINT SYSTEM	2-2
REMOVABLE FRONT PASSENGER SEAT	2-2
DRIVER SEAT STORAGE DRAWER	2-3
FOLD AWAY MIDDLE SEAT	2-3
OPEN DOOR WARNING LIGHT	2-3
INTERIOR FLOOR LIGHTS	2-4
PASSENGER HANDGRIPS	2-4
NON-SKID FLOOR	2-4
III. ACTIVAN EMERGENCY INFORMATION	3-1
A. REAR DOOR EMERGENCY RELEASE	3-1
B. SPARE TIRE, FIRE EXTINGUISHER, FIRST-AID AND TRIANGLE FLARE KITS	3-1
1. SPARE TIRE	3-1
2. FIRE EXTINGUISHER	3-2
3. TRIANGLE FLARE KIT	3-2
4. FIRST-AID KIT	3-2
C. TOWING PRECAUTIONS	3-2
D. JACKING/HOISTING ACTIVAN VEHICLE	3-2
E. BACK-UP RAMP (OPTIONAL)	3-3
IV. ACTIVAN MAINTENANCE	4-1
F. DAILY SAFETY CHECK	4-1
G. FLOOR PANEL CUTTING PRECAUTION	4-1
H. MAINTENANCE SCHEDULE	4-2
I. RICON INSTALLED CIRCUIT BREAKER	4-3
J. RICON MODIFICATION TO FACTORY SYSTEMS	4-3
1. BYPASS OF FACTORY SIDE AIRBAGS	4-3
2. MODIFICATION OF FACTORY AUTOMATIC LEVELING SYSTEM	4-3
V. ACTIVAN SPARE PARTS	5-1

LIST OF FIGURES

<u>Chapter-Figure</u>	<u>Page</u>
2-1 FOLDING RAMP STOWED	2-1
2-2 FOLDING RAMP SWUNG OUT	2-1
2-3 FOLDING RAMP DEPLOYED	2-1
2-4 STORAGE BIN/SPARE TIRE MOUNT	2-1
2-5 STORAGE BIN COMPONENTS.....	2-1
2-6 BACK-UP RAMP IN PLACE.....	2-2
2-7 BACK-UP RAMP STOWAGE LOCATION.....	2-2
2-8 Q'STRAIT ANCHOR POINTS	2-2
2-9 WHEELCHAIR POSITION IN VAN.....	2-2
2-10 REMOVABLE FRONT SEAT	2-2
2-11 FRONT SEAT LOCKED IN PLACE	2-2
2-12 DRIVER SEAT STORAGE DRAWER	2-3
2-13 FOLDAWAY MIDDLE SEAT	2-3
2-14 MIDDLE SEAT UP	2-3
2-15 DOOR-AJAR WARNING LIGHT.....	2-3
2-16 INTERIOR FLOOR LIGHTS	2-4
2-17 PASSENGER HANDGRIPS	2-4
2-18 NON-SKID FLOOR	2-4
3-1 REAR DOOR RELEASE HANDLE	3-1
3-2 VEHICLE CARGO AREA.....	3-1
3-3 SPARE TIRE	3-1
3-4 VEHICLE SUPPORT POINTS.....	3-2
3-5 BACK-UP RAMP IN PLACE.....	3-3
3-6 BACK-UP RAMP ANCHOR SLOTS	3-3
4-1 FLOOR CUTTING PRECAUTION.....	4-1
4-2 RICON INSTALLED INLINE FUSE.....	4-3
5-1 INTERIOR DECALS.....	5-2
5-2 EXTERIOR DECALS	5-3
5-3 ELECTRICAL COMPONENTS	5-4
5-4 DRIVER SIDE FIXED FRONT SEATBASE	5-5
5-5 REMOVABLE FRONT SEATBASE	5-6
5-6 MANUAL FOLDING RAMP ASSEMBLY.....	5-7
5-7 RAMP MOUNTING BRACKET ASSEMBLY	5-8
5-8 RAMP TENSIONER ASSEMBLY.....	5-8
5-9 STORAGE BIN SPARE TIRE MOUNT	5-9
5-10 DRIVER AND PASSENGER FOOTRESTS.....	5-10
5-11 REAR FOOTRESTS	5-10
5-12 CARGO AREA ITEMS	5-10
5-13 ACTIVAN FLARE KITS.....	5-11

LIST OF TABLES

4-1 MAINTENANCE SCHEDULE	4-2
--------------------------------	-----

-Go to next chapter-