



**INSTALLATION INSTRUCTIONS
VEHICLE OPERATING INSTRUCTION DECAL
F8000 SERIES WHEELCHAIR LIFTS
KIT# 01099**

- Read and understand supplemental instructions completely before installation is attempted. Refer to the Service/Owner Manual for additional installation instructions and safety precautions.
- Check contents of kit. Immediately notify Ricon of any missing parts. Claims for damages that have occurred during shipment must be made to the shipper.

TABLE OF CONTENTS	PAGE
I. INTRODUCTION	1
II. INSTRUCTIONS	1

I. INTRODUCTION

One of the most important steps in the F8000 lift installation is to properly affix the operating instruction decals in locations clearly visible to the lift operator and where they will not be damaged. The lift should not be put into use until the operating instruction decals have been properly installed according to the following instructions:

II. INSTRUCTIONS

- TRANSIT MODEL F8000 WHEELCHAIR LIFTS:
 1. Select a smooth and flat area to apply the operating instruction decal, (P/N 26236). The decal must be located where it will be clearly visible to the lift operator from outside the vehicle. The mounting surface must be clean and dry to ensure permanent adhesion.
 2. Peel the backing from decal.
 3. Press the instruction decal face down onto the selected window surface from inside the vehicle. Smooth out any wrinkles and/or trapped air bubbles.
 4. Select a smooth and flat surface near the manual backup system, peel backing, and affix the manual operation decal (P/N 26237). The decal must be located where it will be clearly visible to the lift operator. The mounting surface must be clean and dry to ensure permanent adhesion. Choose a metal or hard plastic area if possible. Smooth out any wrinkles and/or trapped air bubbles.

NOTES